

HOW TO HELP A VICTIM OF SUDDEN CARDIAC ARREST

SCA can happen anywhere, any time. When it does, the heart unexpectedly stops beating properly, and the victim's best chance of survival is immediate CPR and a rescuer with an AED.

Don't panic. You can help.



HOW YOU CAN HELP

Heart Saver CPR — 2015 AHA Guidelines

Check for Danger

Check Responsiveness

Call for Help
Call 911

Check for Breathing

Start CPR

Attach Defibrillator (AED)
as soon as possible and
follow the prompts

HOW ZOLL CAN HELP

ZOLL AED Plus® with Real CPR Help®



Check Responsiveness
"Are you okay?"




Call for Help
Call 911



"Attach AED pads to
patient's bare chest"



"Don't touch patient,
analyzing"

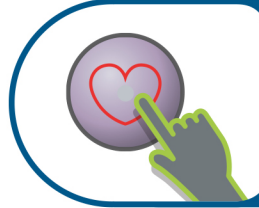

If shock is advised "Press
flashing shock button"



If no shock advised and after
shock delivered "Start CPR"



Not all AEDs are created equal. ZOLL AEDs provide support throughout the CPR process with real-time guidance and feedback on compression rate and depth. This helps responders deliver the best possible CPR to increase a victim's chance of survival.

Distributed By:

SAFT-GARD®

International, Inc.

Phone Order: 1-800-548-GARD (4273) | Fax: 1-888-548-GARD (4273)
Email: sales@saftgard.com | www.saftgard.com

ZOLL®

©2018 ZOLL Medical Corporation. AED Plus, Real CPR Help, and ZOLL are trademarks or registered trademarks of ZOLL Medical Corporation in the United States and/or other countries. All other trademarks are the property of their respective owners. MCN PP 1808 0345

269 Mill Road • Chelmsford, MA 01824 • 978-421-9655 • 800-804-4356 • www.zoll.com/aed